



Rendr Portal Version 2.4 Browser Synchronization

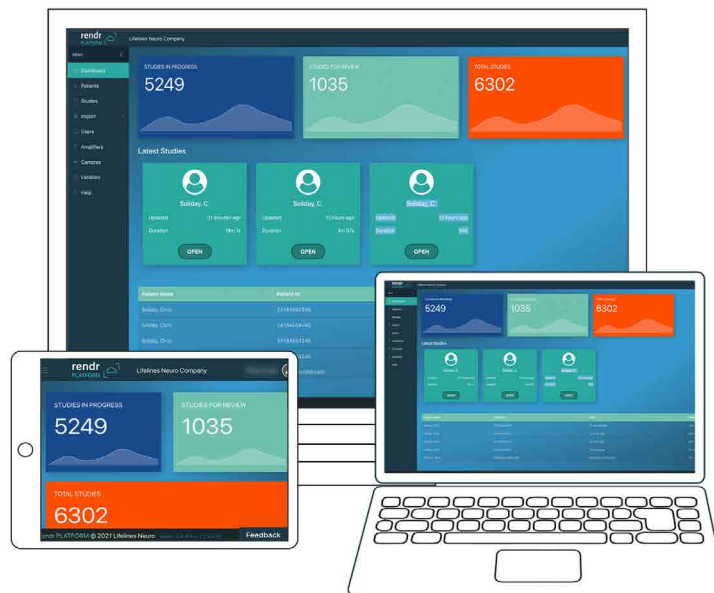



Method 1


First try using CTRL+F5 with the Chrome Browser window open to the Rendr Portal page (rendrneuro.com). If this doesn't work, then follow the steps below.

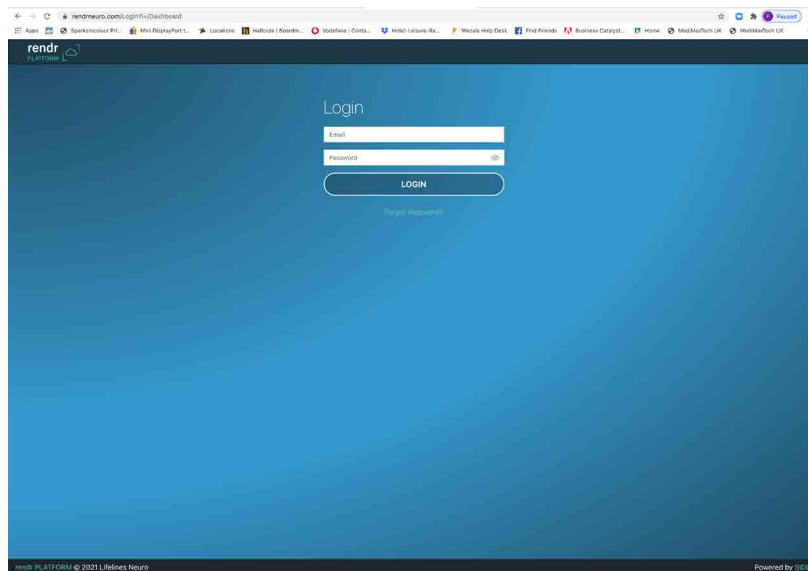

Method 2

Step 1


Open your Chrome browser and navigate to the upper right hand corner of the window and click on the 3 dots:

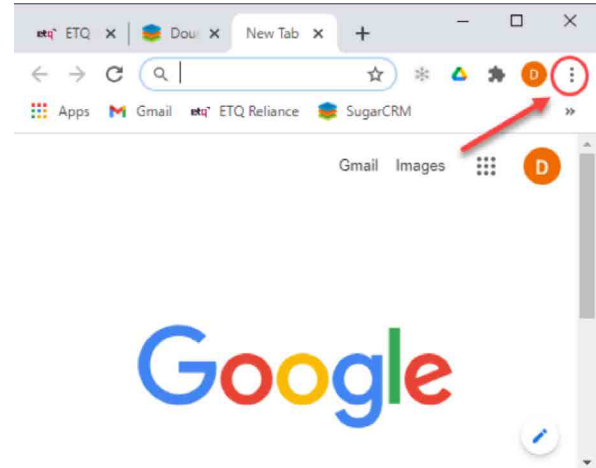

Method 2

Step 2

This will open a menu window and now click on settings:

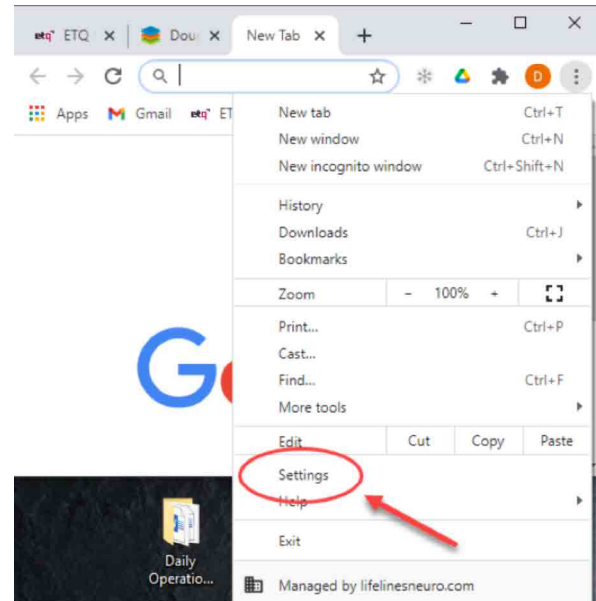




Method 2

Step 3


In settings, scroll down to Privacy & Security and then click on Clear browsing data:

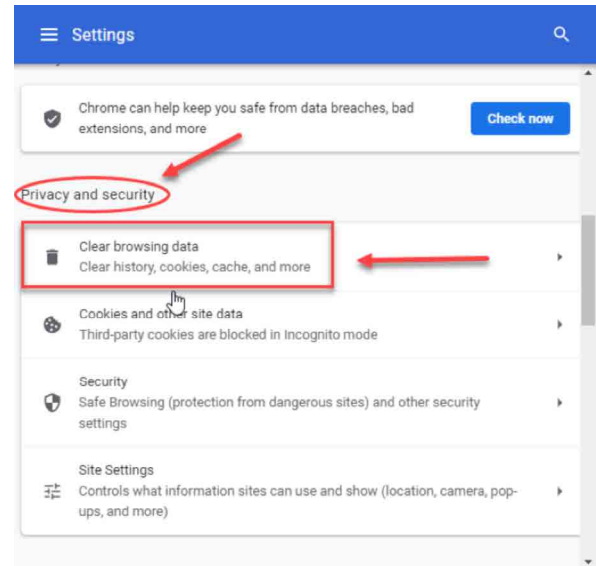

Method 2

Step 4

Once this window opens, click on Clear data and this will clear your cache:

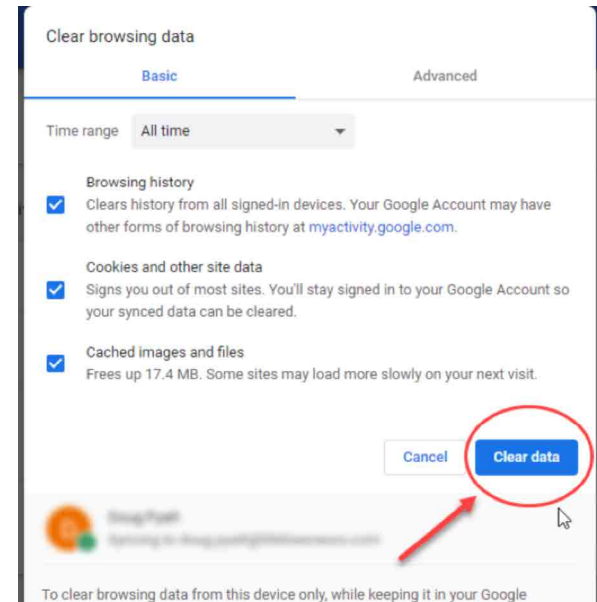



For additional assistance

Tel: 866-889-6505

Email: support@lifelinesneuro.com