

T4A - Formatting SD Card



What You Will Need

SD or micro SD

- Recommended SanDisk Class 10 16GB
- Less than 80mb/sec speed
- Regular or microSD with adaptor



SD Card Formatter

- Located on desktop of Acquisition station or downloaded from: <https://www.sdcardformatter.com/download/>
- **Do not use the Windows Format Tool**, as it may not generate a second BDF file



Formatting an SD Card



To begin formatting:

- Place the SD card into the card reader
- Open the SD card formatting tool from the desktop.

The program may recognize the card automatically.
Make sure that the selected card is the correct one before proceeding, as any data on the card will be lost.

1. If you don't see the drive you want, use the drop down to select the appropriate drive letter.
2. If no card is displayed, click *Refresh*. Then select card.
3. Select *Quick Format*
4. Type a volume label if desired
5. Click *Format*

NOTE: A warning will appear indicating that all the data on the card will be erased. Click yes if you are certain all data can be deleted.

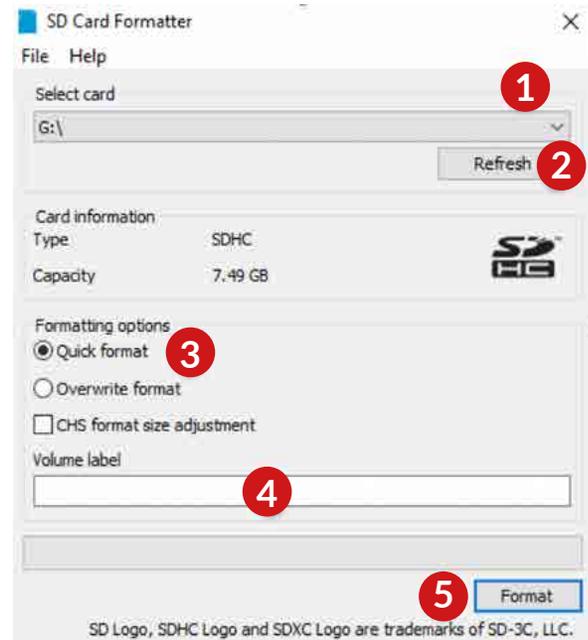
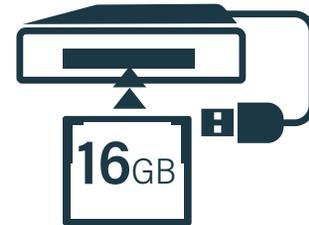




Formatting Completion

- A progress bar will display indicating process.
- Upon completion, a message will display showing successful formatting. Click *OK*.

The card is now ready to be used for recording.

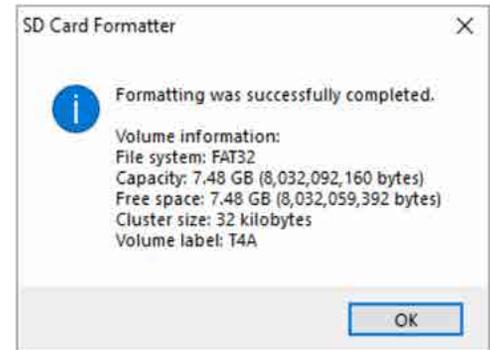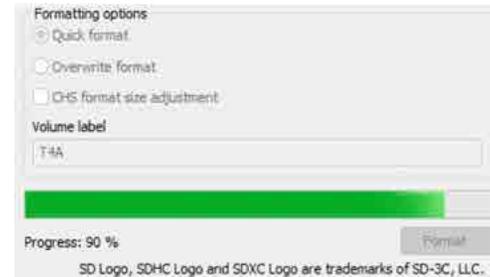



For additional assistance

Tel: 866-889-6505

Email: support@lifelinesneuro.com