



# Trackit Plus Monitoring Quick Guide

Is My Study Recording Properly?

## The Video Recording Window

1. **For Video Recordings:**
  - a. The title of the Video Window should be 'Trackit Video Recording'.
  - b. When **not** recording, the title will be 'Trackit Preview (Not Recording)'
2. **Date/Time** should match the date/time displayed in the Trackit Plus software in the lower-right corner of the program's window. Any difference should be less than 2-3 seconds.
3. **The Video Window** should be the size in this example, and should be recording without freezes, stutters, or other visual defects.
4. **Video Title/ Frame Data**
  - a. Video feed should be labeled with the correct patient's ID.
  - b. The frame-count should be counting upward. Dropped frames are expected but should not exceed 10% of the current total frame-count.







## The Trackit Plus Footer

### 5. Patient ID/Connection Status

- The correct Patient ID should be displayed and should match the video label in **item 4**.
- The bottom field should be displaying '**Online**' and the **amplifier's model**. The amplifier would need to be within Bluetooth-range and connected to the laptop's Bluetooth receiver – or this status will not display.

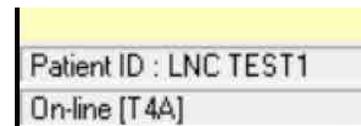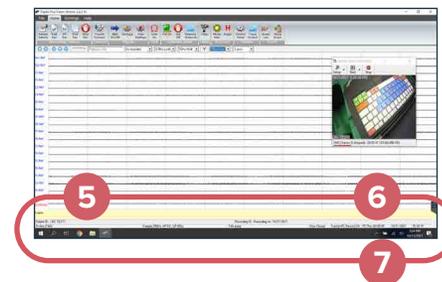

5 Patient ID/Connection Status

### 6. Recording Status

- For Dual Recordings – both **Trackit** and **PC** should be listed as '**Record ON**'. Ambulatory recordings will only display '**Trackit**', and PC recordings will only display '**PC**'.
- PC Rec** – should display the total time that the current PC study has been recording (Only for Dual/PC recordings)
- Date/Time:** Should match system time.



6 Recording Status

### 7. Ensure that the laptop is plugged in, and that the **battery** is charging properly.



7. Battery status

[The Video Recording Window](#)  
[The Trackit Plus Footer](#)  
[The Amplifier Control Panel](#)  
[The Patient Folder](#)  
[For additional assistance](#)


## The Amplifier Control Panel




### 8. Open the Amplifier Control Panel\*



a. In the Trackit A Tab, Verify - Record: On / Flash Card: Yes / Battery: Yes

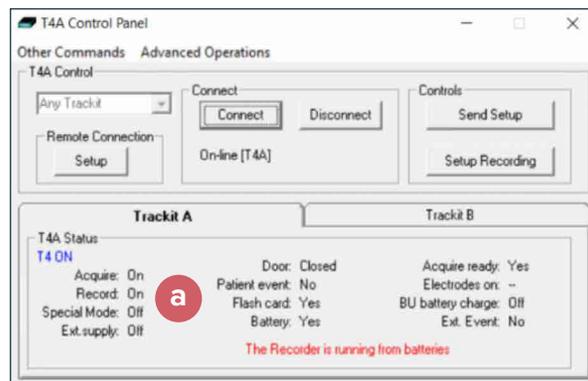

b. In the **Trackit B** Tab, note the values for your logs as needed. The Recording Time should match the number of hours/minutes that the study has been in progress. The battery percentage should be monitored until completion of study.

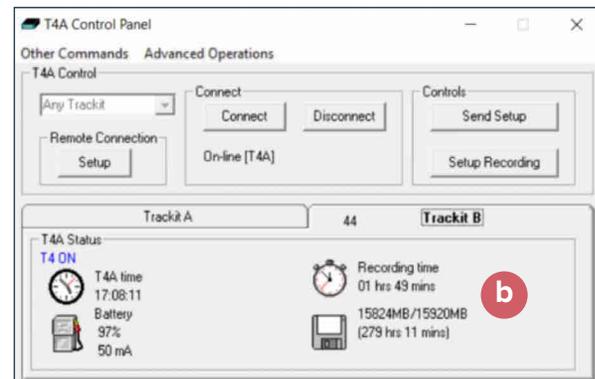

\* PLEASE NOTE: The example here shows the T4A Control Panel, but the Mk3 Control Panel and the process are the same.


Times in this example may not match other examples.



## The Patient Folder

### 9. Patient Folder – Navigate to current Patient’s Folder

- a. For a **PC or Dual Recording** – The following file types: .bdf/ .tev/ .tvx/ should be present and named according to the Patient ID. The BDF’s file-size should increase throughout study – **refresh or re-open the folder to confirm.**
- b. Completed video segments should be of a similar file-size, with one .TVS file per Video File. The most recent video segment should increase in file size.

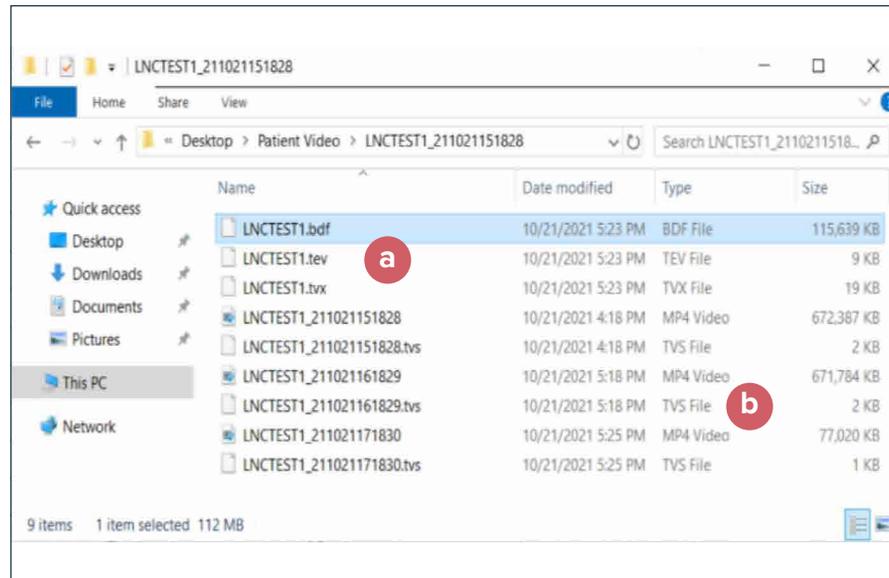

[The Video Recording Window](#)  
[The Trackit Plus Footer](#)  
[The Amplifier Control Panel](#)  
[The Patient Folder](#)  
[For additional assistance](#)

For additional assistance

**Tel: 866-889-6505**

**Email: [support@lifelinesneuro.com](mailto:support@lifelinesneuro.com)**



Imagine EEG Anywhere®

A **SIDIS** Company