

New Features in Trackit Plus v2.8.4.7


FILE MANAGER



File Manager

At the end of a recording, the File Manager window and dialogue is displayed.

The File Manager automatically collects all the relevant associated files for a study and assembles them into appropriate folders on the PC. This makes it much easier for local review and/or the upload of files for remote review to take place.

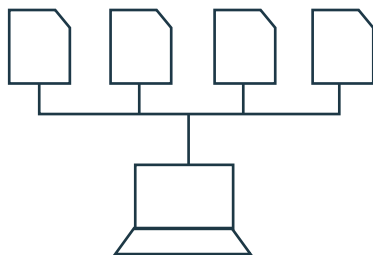

File Manager works for the following recording types:



- Ambulatory (amplifier) recordings with video




- Dual recordings with or without video

File Manager (continued)

File Manager automatically assembles files on the PC as follows:


- **Ambulatory folder,**
e.g. SmithJoh_220311134555

The File Manager assembles the ambulatory files, Events files and all video and synchronization files in this folder.

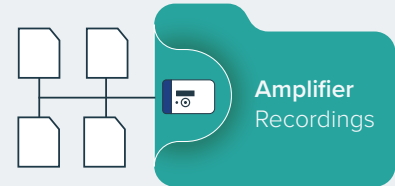


- **PC folder with the [PCFiles] identifier,**
e.g. SmithJoh_220311134555[PCFiles]

The File Manager assembles the PC files into this folder.

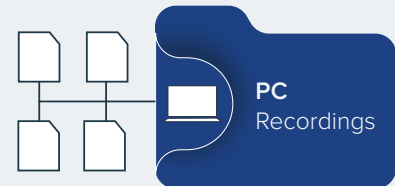

File Manager (continued)

Following Stop Recording, the File Manager steps are as follows:

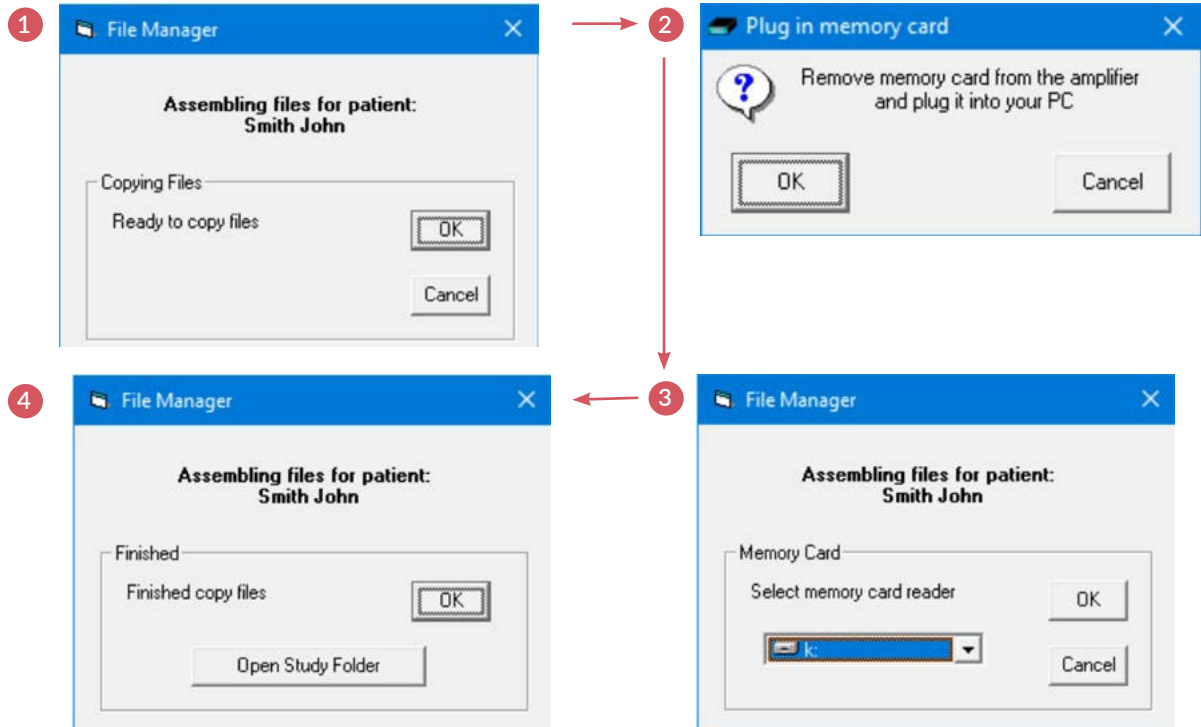

File Manager (continued)


Note: The File Manager can be launched from the main File tab as well as automatically at Stop Recording. The options available here are as follows:

Either

- Proceed as above for the last patient recording

Or via the drop down list:

- Select folder and files for File Manager
- Manually manage files

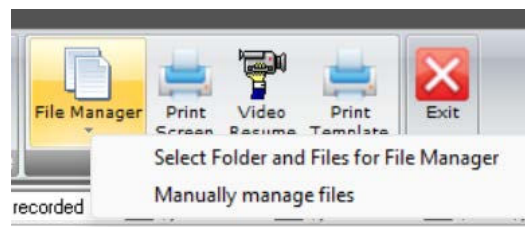



For additional assistance

Tel: 866-889-6505

Email: support@lifelinesneuro.com