

BATTERY MAINTENANCE





Battery Maintenance

The computer on the cart contains up to three removable batteries in separate slots. As with all batteries, these require regular maintenance to ensure optimal performance.

Each month, we recommend moving the batteries through the different slots to ensure they are used evenly.*

NOTE


If using only one battery, this process is not required

Battery Maintenance (continued)

Looking at the back of the computer, the battery slot on the left is Slot 1, the center is Slot 2, and the right is Slot 3.



Battery Maintenance (continued)

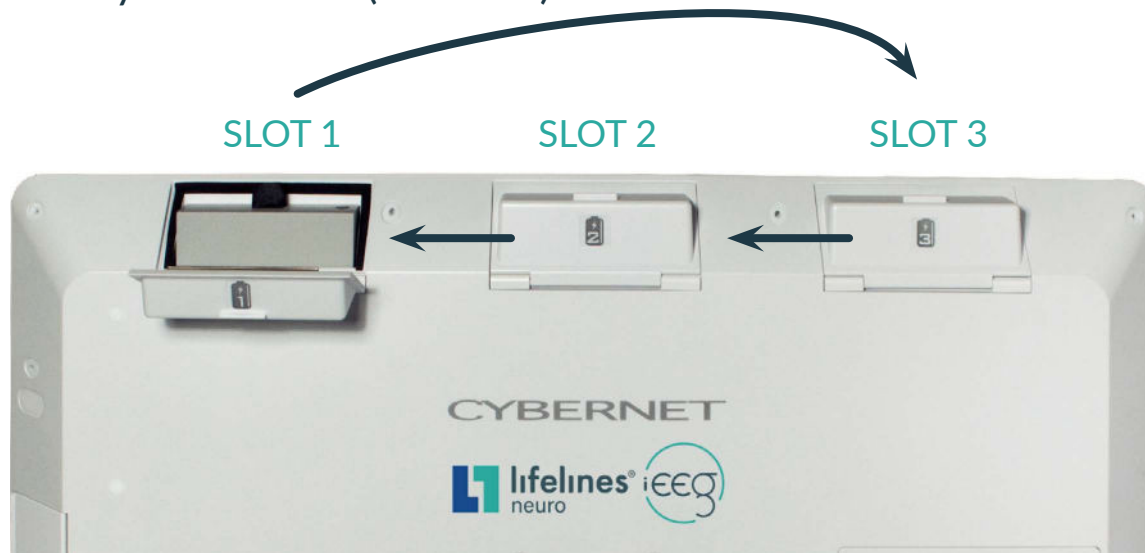

- 1 First, take the battery out of Slot 1.
- 2 Next, move the battery in Slot 2 into Slot 1.
- 3 Move the battery from Slot 3 into Slot 2.
- 4 Last, place the remaining battery into Slot 3.

Battery Maintenance (continued)

Make sure all the doors are closed and confirm that you have three LED status lights on the front of the PC. Each may either be green (charged) or orange (charging), depending on the battery charge level..



Battery Maintenance (continued)

When you have the batteries out, check for any damage or unusual wear. Press the button on the battery to confirm that the status lights all illuminate (if fully charged).





For additional assistance

Tel: 866-889-6505

Email: support@lifelinesneuro.com