

# Quick Start Guide

trackit T4

The Trackit T4 EEG Amplifier is a multi-channel electroencephalograph designed for use in routine EEG and lab monitoring applications. It is available as the T4 32 and T4 68 variants, differing in the number of channels.

- Only use the PC supplied or authorised by Lifelines.
- The user manual, can be found on the Lifelines Neuro website (<https://lifelinesneuro.com/resources>), and must be read before operating the equipment. The supplied PC may be used to provide access to the electronic user guide and a hardcopy can be requested by using the email contact address below.

Free PDF reader software is available, if required, here:

<https://get.adobe.com/uk/reader/>

- Trackit software (version 2.4.1.7 onwards) supports the Trackit T4 and is available from your distributor or Lifelines. Check with them if a newer version is available (or use the email contact address below).
- The software is supported on Microsoft Windows 7, Windows 8 and Windows 10. Windows will automatically install the required USB drivers after connecting the amplifier to the PC for the first time.



The Trackit T4 32 or T4 68 are supplied with:

- A Trackit T4 bag with carry strap
- A Trackit T4 tool, (to eject the SD card).
- The Trackit T4 USB cable.

## Warnings and Cautions

	<p>The Trackit T4 EEG Amplifier must only be used by a healthcare professional, within a hospital or clinical setting, who has the training and knowledge to undertake EEG examinations and is familiar with EEG equipment and practice. The User Manual must be read in its entirety before the equipment is used.</p>
	<p>Do not modify this equipment without the authorization of the manufacturer.</p>
	<p>Federal (USA) law restricts this device to sale by or on the order of a physician.</p>

## Connecting the Trackit T4 Amplifier - Clinical Use

